GR EN

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INTENDED USE:

This product is intended to be used as a waterproof plaster or bandage protector in the shower or bath.

IMPORTANT:

- Before use, lightly file any sharp edges of the plaster that may pierce the protector.
- Hold the plastic ring firmly, and then place the hand or foot that have the plaster or bandage inside.
- Make sure the protector adequately covers the plaster or bandage,
- Dry the plastic ring before removing the protector.
- When removing the protector, hold it by the plastic ring and not by the PVC bag as product failure may be caused
- The product is intended for single patient use.

CAUTION (!):

- Do not use sharp objects in the product.
- Protect the product from abrasions, cuts and holes
- Plastic ring is not designed to have heavy weight applied to it.
- .The protector can become slippery while wet, as the surroundings in the shower area. Use extreme caution when bathing or showering in these conditions.
- Make sure the ring is not too tight in the arm or leg area. Consult your doctor if the ring causes swelling or discomfort.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

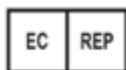
- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the products' components.

Valentine International Ltd.
8th Floor, No. 149, Sec 2, Ta Tung Road
221 Hsichih District, New Taipei City
Taiwan R.O.C.

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Valentine International Ltd.
8th Floor, No. 149, Sec. 2,
Ta Tung Road, 221 Hsichih District,
New Taipei City, Taiwan R.O.C.
Tel: +886.2.8692.6662
Fax: +886.2.8692.6660
info@valentine-dynalife.com

Declaration of Conformity

Manufacturer: Valentine International Ltd.
221 Hsichih District, New Taipei City
Taiwan R.O.C.

SRN: TW-MF-000011689

Authorized Representative: Y. Sung Handelsvertretung
Toulouser Allee 9 40211 Dusseldorf
Deutschland Germany

SRN: DE-AR-000005142

I hereby declare that the products specified above are classified as class 1 devices, as in conformity with the requirements of the EU Medical Device Regulation (EU) MDR 2017/745.


This was verified with conformity evaluation procedures according to Medical Device Regulation Annex VIII.

Products -- Meet requirements for MDR regulation 2017/745

Valentine Item No.	Description	UDIDI
10184-Bariatric	Bariatric Folding Walker	4719879701425
10298LH	Portable Wheelchair Ramp	4719879709308
10563	Fracture Bedpan with Lid	4719879703542
11613/16/WH	Suction Grab Bars	4719879709544
12312	Fold Away Grab Bar	4719879707915
14437	Heel/Elbow Protector	4719879709018
60781/R to 60789/R	Dyna-Seal Cast & Bandage Protectors	4719879706222

This is certify Quality System ISO 13485: 2016,
CERTIFICATED NUMBER 217412-2017-AQ-RGC-NA-PS Rev. 2.0

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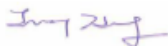Description of symbols that have been printed on the label, Master Carton and or User Manual

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